



Igenity[®] Blood Sample Instructions


Two to three drops of blood are needed on the collector portion of the card. This can be collected from anywhere on the animal. Two locations for easy-to-find blood vessels are the ear and the underside of the tail.



1. Appropriately restrain the animal. Locate a blood vessel, visually or by palpation. Clean the area so the sample is not contaminated with dirt or manure. Use a clean needle or lancet for every animal.



2. Blood can also be sampled with a syringe or blood tube from the vein on the underside of the tail. This may be easier than sampling blood from an ear vein that is covered with long, thick hair.

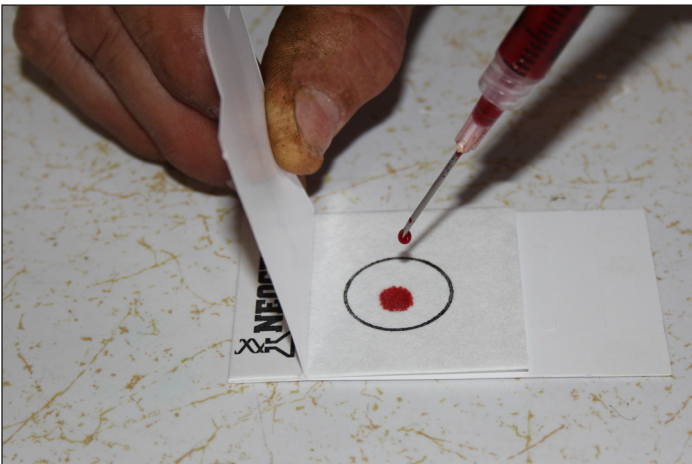

3. Collect two to three drops of blood on the collector portion of the collection card by allowing the blood to drip or squirt onto the card. Do not wipe the needle, ear or tail on the collection card. Discard the needle in an appropriate disposal container.



4. Before placing blood drops on blood cards, write an accurate customer name and animal ID in the spaces provided. Let the cards sit open and air dry before closing cover flap.



Neogen: GeneSeek Operations • 4131 N. 48th Street, Lincoln, NE 68504
877-443-6489 • www.neogen.com/genomics • igenity.support@neogen.com

