


IGENITY[®] BLOOD SAMPLE INSTRUCTIONS

Two to three drops of blood are needed on the collector portion of the card. This can be collected from anywhere on the animal. Two locations for easy-to-find blood vessels are the ear and the underside of the tail.

1. Appropriately restrain the animal. Locate a blood vessel, visually or by palpation. Clean the area so the sample is not contaminated with dirt or manure. Use a clean needle or lancet for every animal.
2. Penetrate the blood vessel with the needle. Let a few drops of blood run uncollected to be sure the sample is not contaminated.
3. Collect two to three drops of blood on the collector portion of the collection card by allowing the blood to drip or squirt onto the card. Do not wipe the needle, ear or tail on the collection card. Discard the needle in an appropriate disposal container.

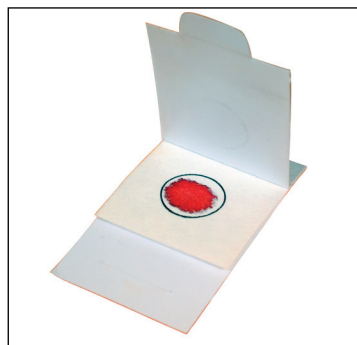

4131 N. 48th St. Lincoln, NE 68504 • 1/877-IGENITY • 402/435-0665
geneseekinfo@neogen.com • www.neogen.com/genomics